

TAiMA



**DIETZ**  
REHA-PRODUKTE

### General information

The rollator described on this information sheet complies with EU Directive 93/42 for medical products. These instructions for use are an integral part of the scope of delivery.

The user should keep them available at all times and keep them with the product if this is passed on to a third party.

The manufacturer reserves the right to make changes relative to the information provided in these instructions as a result of technical developments and improvements.

This document may be reprinted, translated and/or duplicated in any form whatsoever, in part or in full, only subject to the manufacturer's prior written consent.

There is no update service for these instructions for use. Please contact the manufacturer to keep abreast of new developments.

### Specifications and intended purpose

The TAiMA rollator is intended as a walking aid for persons with impaired mobility who weigh up to 150 kg. It also provides a limited seating function for temporary rest.

The rollator should be used only on firm, flat surfaces. It is not intended to be used for any other purpose – in particular as a climbing aid or to carry heavy loads. Any such use can be dangerous!

### Cleaning / maintenance / reuse

**Cleaning**  
Clean and disinfect the rollator at regular intervals. To do so, use mild soapy water with no scouring additives. You can disinfect the rollator using commercially available products. Never use steam degreasers. Never use scouring or caustic cleaning agents.

**Maintenance**  
The TAiMA rollator requires no maintenance. If the braking function weakens, wheel movement becomes stiff or you notice any other functional impairment, please take the rollator to your specialist dealer for servicing.

**Reuse by third parties**  
If you pass on your rollator to a new user, please remember also to hand over all the technical documentation needed to ensure safe operation. In accordance with specifications, the rollator must first be inspected, cleaned and returned to pristine condition by a specialist dealer.

### Safety information

Do not hang bags on the handles as this could cause the rollator to tip over.

Use the service brake (by pulling the brake lever up) on inclines.

Only use the rollator on firm, flat surfaces.

Do not lean too heavily on the handles. Otherwise, the rollator could tip overbackward.

Be careful not to trap your fingers or clothes when folding up the rollator.

Upholstered parts become hot when exposed to direct sunlight and can cause damage to the skin if you touch them. These parts should be covered over. Alternatively, keep your rollator away from direct sunlight.



1. Ergonomic handgrips
2. Brake lever
3. Cane holder
4. Screw for height adjustment
5. Anti folding device
6. Brake

### 1. Setting up/unfolding your rollator

Place the rollator in front of you in the direction of travel and put the parking brake on.

Using both hands, press the tubes on either side of the seat down to the left and right until the folding mechanism clicks into place. Then attach the mesh bag, bag or basket to the appropriate fittings.



### 2. Brakes


**Service brake** (when pushing): Pull the brake lever up.

**Parking brake** (for parking): Press the brake lever down.



### 3. Brake settings

The brake is set correctly if there is no play in either the brake lever or the Bowden cable and if the wheels turn without rubbing or grating. The braking strength can be adjusted using the set screw on the Bowden cable for the brake blocks on the back wheel.

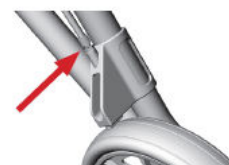

### 4. Curb climber

Pressing your toe down on the extended frame makes it easier to lift the front wheels (to negotiate curbs, for example).



### 5. Grip handle adjustment

Loosen the height adjustment screw and remove it from the frame.

Adjust the push handles to the correct height.


Place the screw in the correct position and tighten it firmly.



### 6. Folding up your rollator

Loose the locking pin under the seat.

Pull the folding strip in the middle of the seat and the rollator will fold up.

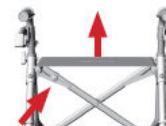

### Technical data

	TAiMA_S	TAiMA_M
Height	78 - 88 cm	79 - 95 cm
Height folded	76 cm	81 cm
Seat height	51 cm	59 cm
Length	63 cm	67 cm
Width	68 cm	63 cm
Width folded	22 cm	22 cm
Max. user weight	150 kg	150 kg
Weight	6,5 kg	6,5 kg
Tyre dimensions	20 x 3,5 cm	20 x 3,5 cm
Grip handles adjustment	5 x 2,5 cm	5 x 4 cm
Meshbag max. weight	5 kg	5 kg

### Scope of delivery

- Rollator TAi MA
- Cane holder for L or R side use
- Shopping bag
- Users manual

### Warranty

The TAiMA rollator has a one year warranty on all parts. Parts subject to wear or misuse are excluded from warranty. Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.